11 in 1 (EMS) 80K SLIMMING MACHINE

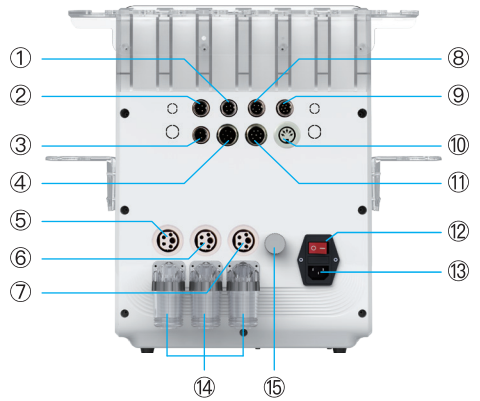
< Instruction Manual >

In order to use this product correctly, please read the manual carefully and keep it properly.

Part name

LCD and operation screen

① 4-pole RF interface

② 3-pole RF interface

③ 80K interface

④ Cold head interface

⑤ Large negative pressure interface

⑥ Medium negative pressure interface

⑦ Small negative pressure interface

⑧ Laser board interface

⑨ 6-pole RF interface

⑩ EMS head interface

⑪ BIO&LED interface

⑫ Switch

⑬ Power interface

⑭ Filter

⑮ Negative pressure regulation

Part name

3 RF Head

4 RF Head

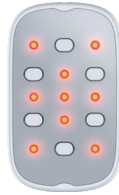
6 RF Head

EMS handle

80K Fat Burn Head

Laser & EMS

Small negative pressure head

Medium negative pressure head

Large negative pressure head

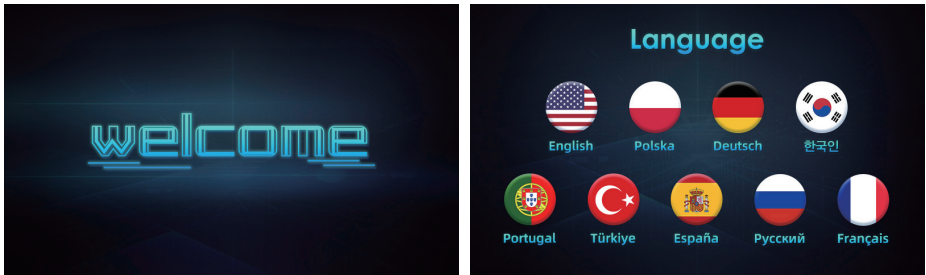
Cold head


BIO&LED

Usage method

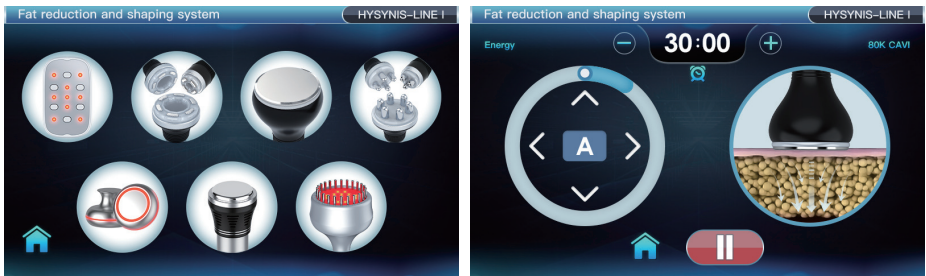1. Connect the function head to the corresponding jack and connect the power cord.
2. Press the power switch, and the indicator light and LCD will light up. At this time, click below to select the required language.

3. Select the work head you want.
4. After selecting, adjust the required working mode, working time, and working energy, click the “▶” button to start working, click the “||” button to pause when using, or click the “🏠” button to return to the working head selection interface.
5. After use, press the “🏠” button and turn off the power.

Function of Different Head

3 RF Head: Suitable for 1–2 Energy

MODE: (A: red light is always on), (B: red light flashes every 1 second), (C: red light flashes every 2 seconds), (D: red light flashes every 3 seconds).
TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the

instrument will stop working.

ENERGY: Adjust the RF energy of the working head. The larger the number, the greater the energy. The 10th gear can be adjusted.

Use for the face or eyes

When use on the skin, it can firms and lifts the skin, reduces wrinkle. Improvement eyes circulation, shrink pores, and puffy eyes. Eliminate wrinkles, dark circles and bags under the eyes. Improve skin texture and eliminate sagging eyes. Rejuvenates the skin and improves eye contour.

4 RF Head: Suitable for 2–3 Energy

MODE: (A: red light is always on), (B: red light flashes every 1 second), (C: red light flashes every 2 seconds), (D: red light flashes every 3 seconds).
TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the

instrument will stop working.

ENERGY: Adjust the RF energy of the working head. The larger the number, the greater the energy. The 10th gear can be adjusted.

Use for the face and neck

Firm skin, enhance skin elasticity, facial wrinkle elimination, skin tightening and lifting, reduce face swelling, reduce redness, relieve eye fatigue, remove dark circles, And eye bags, improve skin quality.

6 RF Head: Suitable for 2-5 Energy

MODE: (A: red light is always on), (B: red light flashes every 1 second), (C: red light flashes every 2 seconds), (D: red light flashes every 3 seconds).
TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the

instrument will stop working.

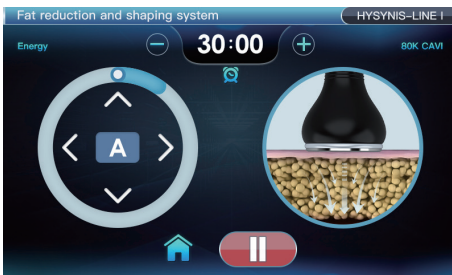
ENERGY: Adjust the RF energy of the working head. The larger the number, the greater the energy. The 10th gear can be adjusted.

Use for the back and waist of Body, thighs and calves

Can tighten the waist, firm and help body shaping. Soften the muscles of the arms and thighs, give the whole body massage and tighten the arms and thighs.

80K Fat Burn Head

MODE: (A: continuous vibration), (B: stop for 1 second per vibration), (C: stop for 2 seconds per vibration), (D: stop for 3 seconds per vibration).

TIME: The default is 30 minutes, adjustable from 0 to

60 minutes. When it is 0, the instrument will stop working.
ENERGY: Adjust the vibration energy of the working head. The larger the number, the greater the energy. The 10th gear can be adjusted.

Use for the body Shaping

It can dissolve fat, detoxify lymph, tighten skin, enhance skin elasticity, reach fat layer directly, have the function of directional RF output, accelerate the movement of fat cells to generate heat energy, and discharge the excess fat and toxin in the face out of the body through sweat glands, liver intestine circulation and lymph circulation, so as to dissolve fat.

Small negative pressure head

MODE: (A: red light is always on), (B: red light flashes every 1 second), (C: red light flashes every 2 seconds), (D: red light flashes every 3 seconds).
TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the

instrument will stop working.

ENERGY: Adjust the RF energy of the working head. The larger the number, the greater the energy. The 10th gear can be adjusted.

SUCTION: Adjust the suction time of negative pressure. The larger the number is, the longer the suction time will be. It is invalid when it is 0.

RELEASE: Adjust the suction time of negative pressure. The larger the number is, the longer the suction time is. It is invalid when it is 0.

Note: The inspiratory and outgassing intervals are only available when the number is adjusted at the same time.

Use for the face or eyes

Facial lifting, firming, anti-aging, relieving pain, scraping and cupping.

Improve skin sagging caused by fat and skin aging. Help eliminate waste materials in the body and promote metabolism. Negative pressure massage function can promote fat cell volume reduction. Vacuum negative pressure massage can fully absorb various skin tissues between the multi-polar thermal melting radio frequency, so that the radio frequency energy can reach the deep skin tissue more effectively. At the same time, the adsorption of negative pressure promotes blood circulation and metabolism. Let the muscle tissue be fully relaxed and relieve the fatigue of the skin and muscle tissue.

Medium negative pressure head

MODE: (A: red light is always on), (B: red light flashes every 1 second), (C: red light flashes every 2 seconds), (D: red light flashes every 3 seconds).

TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the

instrument will stop working.

ENERGY: Adjust the RF energy of the working head. The larger the number, the greater the energy. The 10th gear can be adjusted.

SUCTION: Adjust the suction time of negative pressure. The larger the number is, the longer the suction time will be. It is invalid when it is 0.

RELEASE: Adjust the suction time of negative pressure. The larger the number is, the longer the suction time is. It is invalid when it is 0.

Note: The inspiratory and outgassing intervals are only available when the number is adjusted at the same time.

Use for the face and neck

Shoulder and neck scraping therapy for lymphatic detoxification relieving pain, scraping and cupping.

Improve skin sagging caused by fat and skin aging. Help eliminate waste materials in the body and promote metabolism. Negative pressure massage function can promote fat cell volume reduction. Vacuum negative pressure massage can fully absorb various skin tissues between the multi-polar thermal melting radio frequency, so that the radio frequency energy can reach the deep skin tissue more effectively. At the same time, the adsorption of negative pressure promotes blood circulation and metabolism. Let the muscle tissue be fully relaxed and relieve the fatigue of the skin and muscle tissue.

Medium negative pressure head

MODE: (A: red light is always on), (B: red light flashes every 1 second), (C: red light flashes every 2 seconds), (D: red light flashes every 3 seconds).

TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the

instrument will stop working.

ENERGY: Adjust the RF energy of the working head. The larger the number, the greater the energy. The 10th gear can be adjusted.

SUCTION: Adjust the suction time of negative pressure. The larger the number is, the longer the suction time will be. It is invalid when it is 0.

RELEASE: Adjust the suction time of negative pressure. The larger the number is, the longer the suction time is. It is invalid when it is 0.

Note: The inspiratory and outgassing intervals are only available when the number is adjusted at the same time.

Body maintenance

Improve skin sagging caused by fat and skin aging. Help eliminate waste materials in the body and promote metabolism. Negative pressure massage function can promote fat cell volume reduction. Vacuum negative pressure massage can fully absorb various skin tissues between the multi-polar thermal melting radio frequency, so that the radio frequency energy can reach the deep skin tissue more effectively. At the same time, the adsorption of negative pressure promotes blood circulation and metabolism. Let the muscle tissue be fully relaxed and relieve the fatigue of the skin and muscle tissue.

Laser & EMS

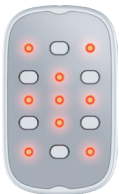
MODE: (A: red light is always on), (B: red light flashes every 1 second), (C: red light flashes every 2 seconds), (D: red light flashes every 3 seconds).
TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the

instrument will stop working.

ENERGY: Adjust the micro-electric energy of the working head. The larger the number, the greater the energy. The 10th gear can be adjusted.

LIGHT: Adjust the brightness of the laser panel lamp. The greater the number, the greater the brightness. The 10th gear can be adjusted.

Use for the body Shaping

The Following information is provided for the correct utilization of IPL Skin treatment system. the Information includes not only the accident protection regulations the products comply with, but also the effective regarding proper use of the products.

Cold head

MODE: By default, only mode A is available.

TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the instrument will stop working.

ENERGY: Adjust the working energy of the working head.

The greater the number, the greater the energy. The 10 gears can be adjusted.

1. Help skin firmness, maintain elasticity, prevent allergies and other problems;
2. Shrink pores, lock water and reduce wrinkles;

BIO&LED

MODE: There are four (A/B/C/D) micro-electric massage modes, which can be selected according to personal preferences.

TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the

instrument will stop working.

ENERGY: Adjust the micro-electric energy of the working head.

The larger the number, the greater the energy. The 10th gear can be adjusted.

LED COLOUR: Adjust the light color. The light sequence is red light, green light, blue light, orange light, yellow light, purple light, blue light, circular light.

LIGHT: Adjust the light brightness, The 10 gears can be adjusted.

Application : forehead, eyes' around, cheek, mouth, double chin, neck, body
Microcurrent–safe and painless facial helps erase fine lines and wrinkles, while firming your skin and defining your features

EMS head

MODE: There are four (A/B/C/D) micro–electric massage modes, which can be selected according to personal preferences.
TIME: The default is 30 minutes, adjustable from 0 to 60 minutes. When it is 0, the instrument will

stop working.

ENERGY: Adjust the micro–electric energy of the working head. The larger the number, the greater the energy. The 10th gear can be adjusted.

HOT:Adjust the working head temperature, adjustable at 10 gears.

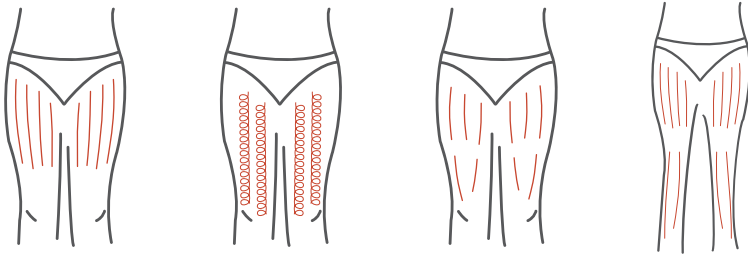
It is a non–invasive slimming treatment technology that can burn fat while exercising. Just like completing 20,000 sit–ups or squats in just 30 minutes, every 2 to 3 weeks, during 4 to 6 treatments, average fat decreases and muscle mass increases. The treatment area includes the abdomen, buttocks, arms and thighs. safe and comfortable treatment

Using Guidance


Thigh

- (1) From bottom to top, push to groin to dredge lymph;
- (2) Turning anticlockwise from bottom to top can help break down fat;
- (3) Decompose greasy by pushing back and forth;
- (4) Pull down from the knee to the groin to improve the curve.


Hips

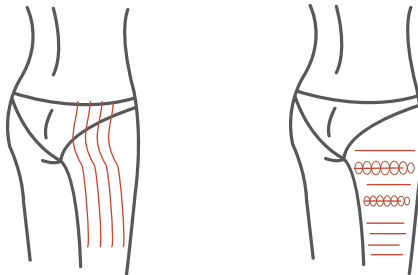- (1) Pull to the waist in the direction of lymph;
- (2) Circle anticlockwise from bottom to top and move back and forth;
- (3) According to the muscle texture, it can slide and compact the buttock skin, making the buttock plump and firm, presenting the perfect curve.

Back

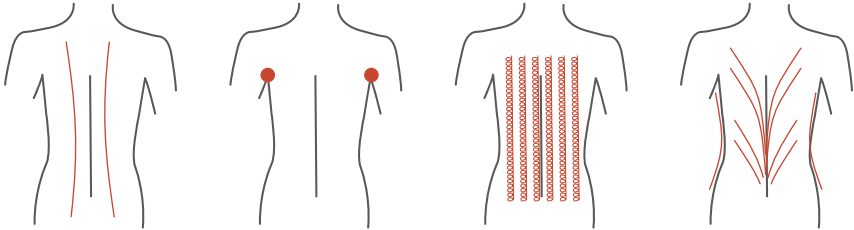- (1) Push back and forth 2–3 times;
- (2) Circle anticlockwise to stimulate the lymph nodes;
- (3) On the back, turn anticlockwise and move up and down to help break down fat;
- (4) Using lymphatic drainage technique to pull toxin to lymph nodes can help improve back curve.

Arm

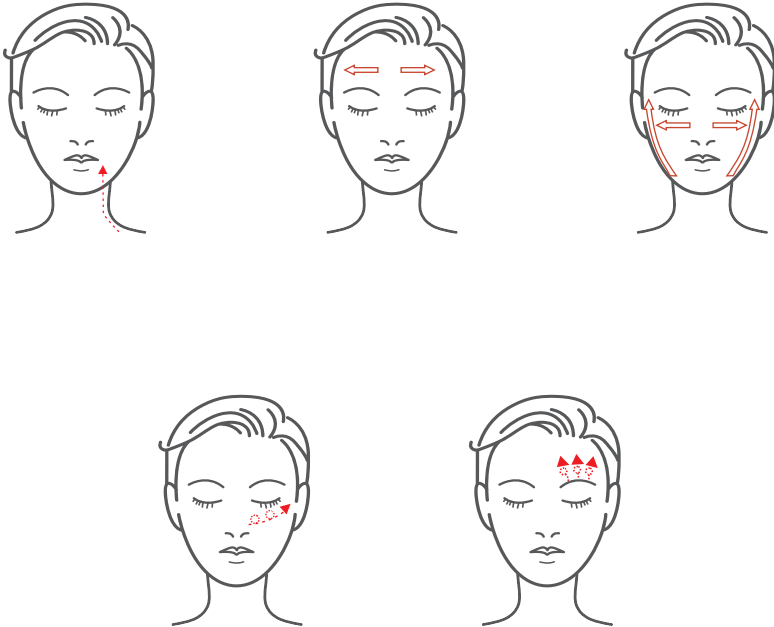
- (1) From the inner elbow to the armpit, do lymphatic drainage;
- (2) To stimulate lymph nodes in a circle;
- (3) Break down excess fat by turning it anticlockwise or pushing it back and forth;
- (4) Tighten skin from elbow to armpit.

Face



(1) Evenly apply the product on the face, use the face RF head, operate from the bottom to the top to lift the face, make the skin in a tense state, it is recommended to use for 5 minutes;

(2) Apply the product evenly to the face, use the RF head on the face, operate in the direction of the arrow, it is recommended to use it for 5–8 minutes.

Abdomen

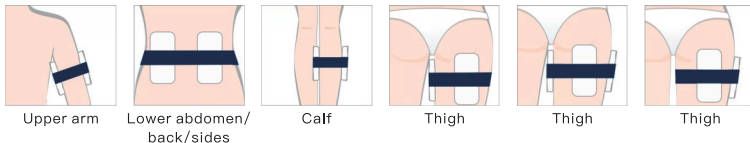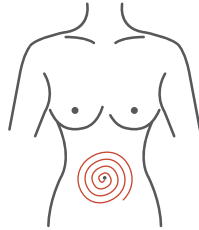- (1) Around the navel from small to large, clockwise circle to help large intestine peristalsis;
- (2) From the small circle to the large circle, push it anticlockwise.

Warm Tips for You

1. Do not use on your original and dry skin surface, it will burn your skin. Make sure you apply cream or oils to your skin then you can use.
2. When using, there must be an aqueous medium (essential oil/creams, etc.), otherwise it will damage to the machine Do not use on the same part for too long, and avoid body parts with bones.
3. Do not connect the power until probe is well installed.
4. The machine is not allowed to replace probe when in the charging state.

5. Please take off the metal jewelry when using the machine.
6. When the machine is used for 1 hour, please stop for 10 minutes to let the machine rest.
7. Make sure you turn off the power after using, and clean the probe keep good maintenance for next time use.
8. Please keep the probe clean every time, use a dry cloth to wipe it, do not use water to clean.
9. After using, clean you skin and apply hydrating mask or creams, oils after 10 minutes.
10. When using 80KHead, some people may experience tinnitus. This is normal because this sound wave exceeds the range of human sound waves.

Warranty & After Service

We Promise that all the machine are experiences a strict inspection by us to ensure the best quality. We will use our professional skills to deal with your problem.

Hope you can join us on this journey together.

If you have any question or other problems about our products, please feel free to contact us without hesitation. We always by your sides.

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