
40K blasting fat R.F multipolar slimming instrument

Part name :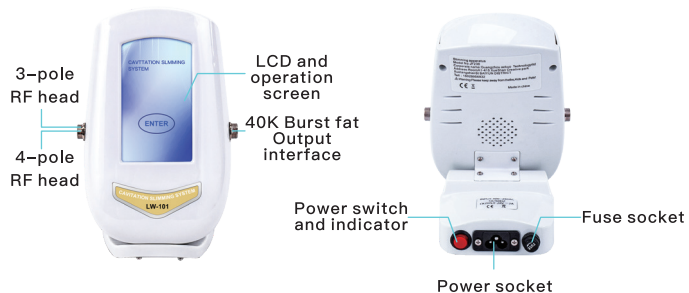

Operating instructions:

1. Connect the handle to the corresponding jack and connect the power cord;
2. Press the power switch, the indicator light and LCD will be on;
3. Click the "enter" key in the LCD (as shown in Figure 1) to enter the operation mode (as shown in Figure 2) and then select the required function;
4. Touch the 40K key to display figure 3, adjust the required mode, working time and working intensity, press the "start" key, the 40K grease blasting head starts to work, and other R.F function selection methods are the same;
5. After use, press the "back" key to return to the interface shown in Figure 1, and then turn off the power supply.

Function and operation principle:

1、40KHZ Fat blasting head

The 4000Hz sound wave emitted by the explosive head with strong sound wave can make the fat cells of human body produce a strong impact after entering the human body, which can effectively consume heat, water and burst the fat cells, reduce the fat cells and achieve the effect of removing fat.

Main efficacy

- a. Lose weight in whole or in part;
- b. Increase the speed of metabolism, strengthen the elimination of waste and moisture;
- c. Repair of stria gravidarum;
- d. Relax muscle, relieve muscle spasm, relieve muscle pain symptoms;
- e. Tighten the muscles of arms, feet, thighs, buttocks, back and abdomen to reshape the body;
- f. It can effectively improve the orange skin of buttocks and thighs, and solve the problem of postpartum abdominal relaxation.

2、4 pole RF head

It can dissolve fat, detoxify lymph, tighten skin, enhance skin elasticity, reach fat layer directly, have the function of directional RF output, accelerate the movement of fat cells to generate heat energy, and discharge the excess fat and toxin in the body out of the body through sweat glands, liver intestine circulation and lymph circulation, so as to dissolve fat.

3、3 pole RF head

When it acts on the skin, it can heat the collagen fiber of the dermis to 45–65 degrees, the collagen fiber will contract immediately, and the loose skin wrinkles will be tightened. At the same time, the stimulated collagen hyperplasia of the dermis will increase the thickness and density of the dermis, so as to remove wrinkles, cut scars, restore the elasticity and luster of the skin to achieve the purpose of tightening and removing wrinkles.

Scope of application:

- a. Arms, calves, thighs, arms, back, abdomen and other parts;
- b. It can improve and treat the adipose tissue in various conditions;
- c. Shushou, decompression and Shushou pain (such as arthralgia and sciatica) can promote the mechanical aerobic movement of skin tissue, relieve the pressure quickly and eliminate fatigue;
- d. Enhance and tighten skin, restore skin elasticity;
- e. Activate cell circulation, promote metabolism, produce more collagen and elastic fiber, improve skin tissue of face and body;


Operation method:

For example: for the operation of body parts (excluding the face), use the fat blasting head and RF head, and operate in the following order: Before operation, clean the skin first, use the essential oil or gel to smear evenly on the skin surface, then massage.

Using Guidance


Thigh

- (1) From bottom to top, push to groin to dredge lymph;
- (2) Turning anticlockwise from bottom to top can help break down fat;
- (3) Decompose greasy by pushing back and forth;
- (4) Pull down from the knee to the groin to improve the curve.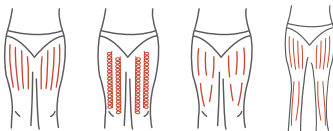


Hips

- (1) Pull to the waist in the direction of lymph;
- (2) Circle anticlockwise from bottom to top and move back and forth;
- (3) According to the muscle texture, it can slide and compact the buttock skin, making the buttock plump and firm, presenting the perfect curve.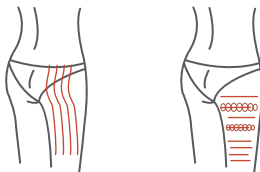


Back

- (1) Push back and forth 2–3 times;
- (2) Circle anticlockwise to stimulate the lymph nodes;
- (3) On the back, turn anticlockwise and move up and down to help break down fat;
- (4) Using lymphatic drainage technique to pull toxin to lymph nodes can help improve back curve.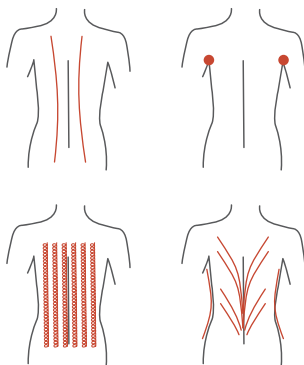


Arm

- (1) From the inner elbow to the armpit, do lymphatic drainage;
- (2) To stimulate lymph nodes in a circle;
- (3) Break down excess fat by turning it anticlockwise or pushing it back and forth;
- (4) Tighten skin from elbow to armpit.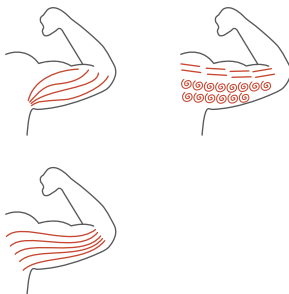


Abdomen

- (1) Around the navel from small to large, clockwise circle to help large intestine peristalsis;
- (2) From the small circle to the large circle, push it anticlockwise.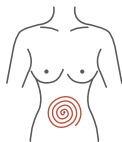

Face

- (1) Evenly apply the product on the face, use the face RF head, operate from the bottom to the top to lift the face, make the skin in a tense state, it is recommended to use for 5 minutes;
- (2) Apply the product evenly to the face, use the RF head on the face, operate in the direction of the arrow, it is recommended to use it for 5–8 minutes.

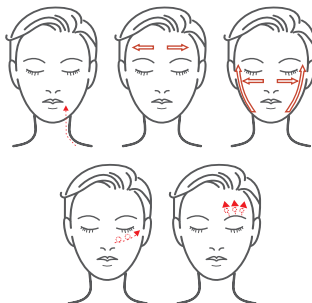Unsuitable people :

1. If there is a wound on the skin, avoid the wound or not;
2. Those suffering from leprosy are forbidden to do so;
3. It is forbidden for patients with heart disease (such as arrhythmia), especially those with pacemaker;
4. Cancer, cancer patients are forbidden to do;
5. Patients with acute heart phlegm or infectious diseases are forbidden to do so;
6. Pregnant women, breast-feeding women, forbidden to do during menstruation;
7. Serious diseases, such as heart disease, gastric ulcer, severe gastric disease, duodenal ulcer, hypertension, abdominal operation less than three months, diabetes, etc;
8. Metal and silica gel are forbidden in vivo;
9. During operation, the handle must be 90 degrees vertical to the skin to avoid discomfort to the guest;
10. Before starting the operation, first contact the probe to the skin vertically, then turn on the host switch, adjust the operation mode energy to the minimum, and then adjust the intensity according to the customer's skin bearing capacity;

Precautions for use :

- 1, first clean the skin, then use a special gel or weight loss essential oil.
2. There must be water medium (such as essential oil, etc.) in the treatment, otherwise it will cause certain loss to the instrument;
3. The operation head does not stay in one place for a long time, avoiding places with bones;
4. Do not use disinfectant product to disinfect, suggest to use wet cotton or towel;
5. The operator and the person receiving the treatment take off the metal decorations on the body;
6. If the machine is used continuously for 1 hour, please stop for 10 minutes, and then use it again;
7. Tinnitus may occur to some of the guests when using the fat blasting head, which is a normal phenomenon, because this wave band has exceeded the range of human acoustic wave.

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