MINI 40K

Fat-blasting Weight-reducing Device

To reduce your weight professionally

USER'S MANUAL

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About Product

Dear customer,thank you for purchasing this weight-reducing device!

We believe this device will offer you a professional way of weight-losing and help you shake off extra fat effectively.As its name called,40K indicates the device's working frequency is 40000Hz which technically is the ultrasonic that inafrequency band which is very ideal for reducing weight.

Safe to use as it is,you still need to read the user manual through before operations.

CONTENTS

Product theory & theory	1
Product details	4
Use guide	6
Advised use guide	8
Device safety concerns	9
Cautions on treatment wand	10
Tips	11
After sale service	12

PRODUCT WORKING THEORY & EFFICACY

The fat-blasting device comes with the fat reducing theory of ultrasonic which produces high density energy. When the ultrasonic penetrates deeply to skin, the ultra wave can alienate fat cells to cause voids, these voids will implode to release the accumulated ultra sonic energy and trigger ultra fast micro vibration to burn fat instantly. This weight losing method enjoys the feature that zero hurts to skin tissues and lymph. Also this device helps user to tighten and lift skin to shape you a dream figure.


PRODUCT EFFICACY

85% resolved fat will be channeled out of body through lymph system,
15% fat will be translated into sugar via liver to support man's daily activities.

- 1.This weight reducing device can be a alternative of liposuction.
- 2.This device helps in tightening and lifting skin,also promote skin blood circulation.
- 3.Helps users to relax and massage their skin and muscles.
- 4.Effective in reducing weight and resolve fat under skin.
- 5.Effective to remove cellulite and helps parturient to solve problems like sagging belly after childbirth.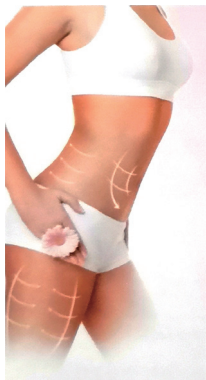

Shape you the attractive figure

Lift and tighten skin
Skin tightening
Refine skin
Belly care
Lift and shape hip

Efficacy

- 1.Waist
Promote peristole and ovarian function.
- 2.Hip
Shape hip and tighten Skin.

Efficacy comparison

Shape waist and legs

Device technology

- 1.world class fat blasting technology.
- 2.Applicable for all skin types.
- 3.Painless,zero skin hurts,no skin tissue damages.
- 4.Easy operation,convenient to use.
- 5.Effective and safe.

Device features

- 1.Reduce weight without surgery.
- 2.No hurt and bleeding.
- 3.Zero adverse effects.
- 4.No damages to skin tissues.
- 5.Easy to operate.

PRODUCT STRUCTURE SHOW

Product parameters

Frequency:40--60Hz

Energy output:35--60J/cm²

Power:60W


Weight:2.4Kg

Product size:280*280*130mm

Power input:100--240V 2A(MAX)

Device addapter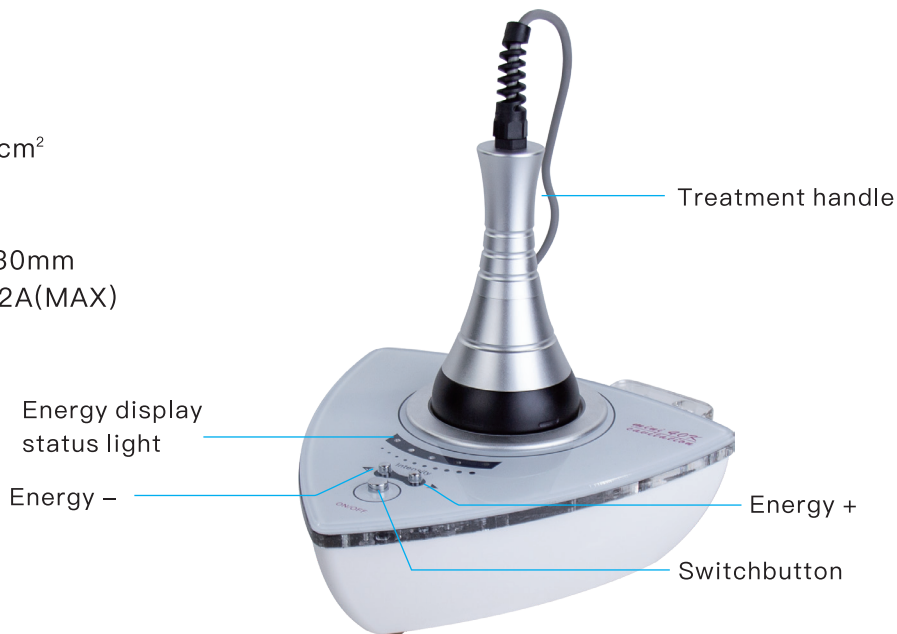

THE BACK OF THE PRODUCT STRUCTURE


USE GUIDE


1. Please store this device in cool place and make sure the heat emission fan at the bottom of the device has enough space for ventilating heat.
2. Please carefully and rightly connect the treatment handle.
3. The red switch button lies at the back of the device . “I” means the device is on, “O” means the device is off.
4. Long press the ON/OFF button for 2 seconds, then the third light will be on, that means user at the moment can be adjustable.
5. The 40K fat-blasting head's max continuous working time is 30 minutes, after 30 minutes using, please stop using for about 5 minutes to cool down the surface temperature of the treatment head.
This device has been preset before delivery.
6. After using, please switch off the device and unplug it.
7. After use according to the “OFF” button on the surface of the instrument and equipment at the back of the red button “O” to shut down the instrument, then unplug the power cord, etc.

ADVISED USE GUIDE


Before use the device, please smear some skin care gel on the target skin area. Then adjust the proper device working intensity and follow the instructions below.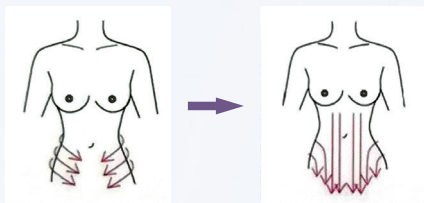


Glide the treatment head slowly and gently by following skin texture from belly to groin. Please follow the instructions showed in the picture.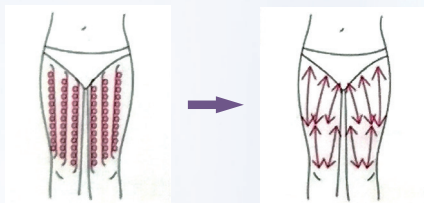

Glide the treatment head slowly from bottom to the top and repeat this for several times. Please follow the instructions showed in the picture.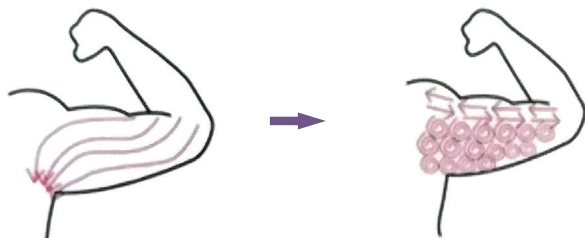

First,glide the treatment head from the elbow to the armpit and glide in circle.
Then do this movement in reverse direction and repeat the whole process for several times.

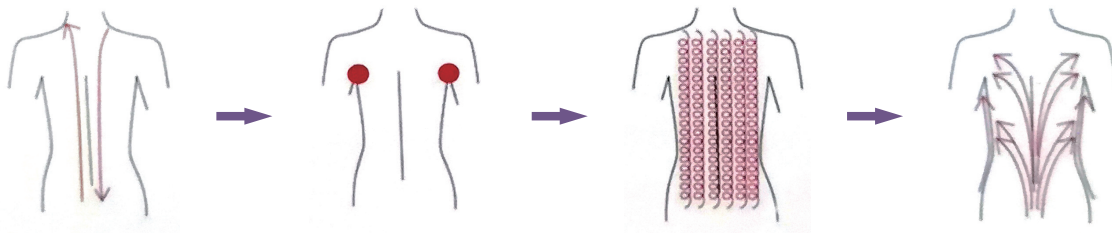First,glide the treatment head slowly and gently from the upper part of the body to the bladder and repeat this process for several times.
Then glide the treatment in circle to burn the fat inside body.Next glide the treatment by following the directions showed in above picture to promote the lymph circulation.

SAFETY CONCERNS

- 1.This device belongs to a sophisticated kind which should be kept dry and cool.
 - 2.Prevent this device from any violent impacts.
 - 3.Please clean this device frequently to ensure a normal and smooth running.
 - 4.If the device is not in use,please cover it by using a cloth cover to keep it from dust.
- When the device is left unused for a long time,please plug the device every 15 days to keep the device from damages caused by humidity.

USE PROHIBITED

- 1.Device's cable is damaged or find the cable is overheated.
- 2.Device is not tightly connected to the power socket.
- 3.Do not bend the device's plug forcefully.
- 4.Do not unplug the device during device using.
- 5.Do not use incompatible power cable.
- 6.Make sure the device is unreachable for kids.
- 7.Do not clean the device by using any corrosive liquid.
- 8.Do not operate the device when hands are wet.

CAUTIONS ON USING TREATMENT HEAD

1. Make sure the treatment head fully contact the target skin, better vertically.
2. Glide the head slowly and gently.
3. After finish using, please switch off the function of this treatment head.
4. Any damages found on the electron of this treatment head.

REMINDER

- 1.The ultrasonic produced by this device may cause uncomfortable tinnitus which is normal during this device using.
- 2.Please attach enough attention to the maintenance of the device,the sophisticated units inside especially.
- 3.Loosened device components connections may result in short circuit.
- 4.After device using,please clean it rightly for next time use.
- 5.Patient who has skin wounds or injury are forbidden to take the treatment.
- 6.Diseases suffers like epilepsy,heart disease or diabetes are forbidden to take the treatment.
- 7.Pregnant women,women in period or suckling period are forbidden to take the treatment.

AFTER SALE SERVICE

This device(main body of this device)enjoys 6 months warranty since the day of purchase.
Any device(main body of this device)problems detected in 3 months,charge-free repair service is offered,repair fee is redquired over 3 months.

The device accessories enjoy 1 month charge-free repair for surfaced quality problems, repair fee is required over 1 month.

Following device malfunctions are not included in the range of charge-free repair service.

- 1.Dismantle or alter the device by users themselves.
- 2.Accidentally fall during device using.
- 3.Device malfunctions caused by poor maintenance.
- 4.Operate the device without following the product manual.

Service card

Product name:_____ Date of purchase:_____ Buyer'sname: _____

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